## Cleaning Tinkerine Face Shield Lite

### Step 1. Precautions for cleaning

- Before you begin, protect yourself by wearing gloves, a face mask, and another face shield if possible.
- Clean and disinfect the working surface thoroughly.
- Only clean one face shield at a time to avoid cross-contamination.

### Step 2. Disassembling the face shield


- Disassemble the face shield in the reverse order of the assembly:
  - Remove the knit elastic band
  - Remove the clips
  - Remove the forehead strap
- Due to the porosity of the knit elastic band and the 3D printed clips, we cannot guarantee that these parts are fully sanitizable after being used. We recommend that they are replaced regularly at the user's discretion. They should be replaced immediately if damaged or upon direct contact with anyone with flu-like symptoms.

### Step 3. Cleaning the PETG visor and the forehead strap

- It is recommended to fully submerge the PETG visor and the forehead strap in a disinfectant solution to ensure complete sanitization before and after every use.
- After removing the visor and the strap from the solution, allow a few minutes for the parts to dry. Wipe down any remaining disinfectant droplets with a clean paper towel.
- If submerging is impossible, thoroughly spraying the visor and the strap with a disinfectant can also be effective. Allow a few minutes to dry. Wipe down any remaining droplets.
- Our recommended disinfectants are:
  - Isopropyl alcohol (70%)
  - Hydrogen peroxide (28%)
- To find out more about specific cleaning products, please refer to the [list of disinfectants approved by Health Canada](#).

### Step 4. Reassembling the face shield

To reassemble to face shield, use the disinfected PETG visor and forehead strap with a set of new replacement parts. Refer to the assembly manual.

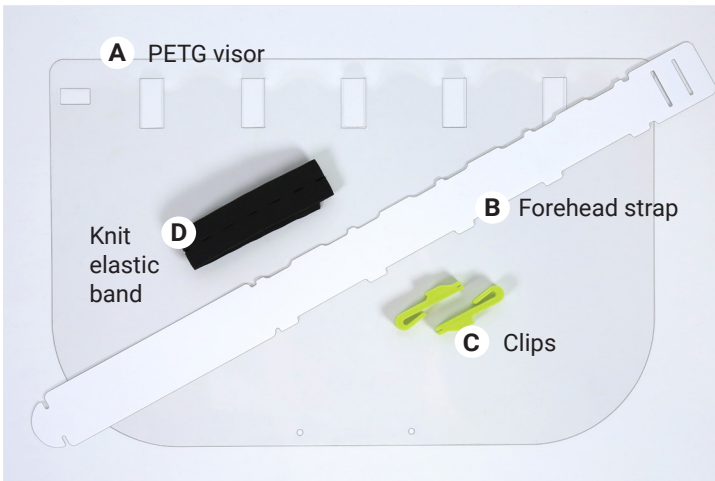Tinkerine Face Shield Lite is not sterile. Since the conditions and methods of assembly and use of Tinkerine Face Shield Lite are beyond the manufacturer's control, the manufacturer expressly disclaim any and all liability arising from any assembly and use of Tinkerine Face Shield Lite. NO WARRANTY OF FITNESS FOR ANY PARTICULAR PURPOSE, WARRANTY OF MERCHANTABILITY OR ANY OTHER WARRANTY, EXPRESSED OR IMPLIED, IS MADE CONCERNING TINKERINE FACE SHIELD LITE. The user should thoroughly test any application before use. By assembling and using Tinkerine Face Shield Lite, the user does so at their own risk and waives all claims against the manufacturer. The user releases the manufacturer from all liability, whether due to foreseeable or unforeseeable risks, relating to Tinkerine Face Shield Lite. The user is solely responsible for their assembly and use of the Tinkerine Face Shield Lite. The manufacturer is not responsible for errors or omissions relating to Tinkerine Face Shield Lite. The manufacturer does not provide any medical advice.

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