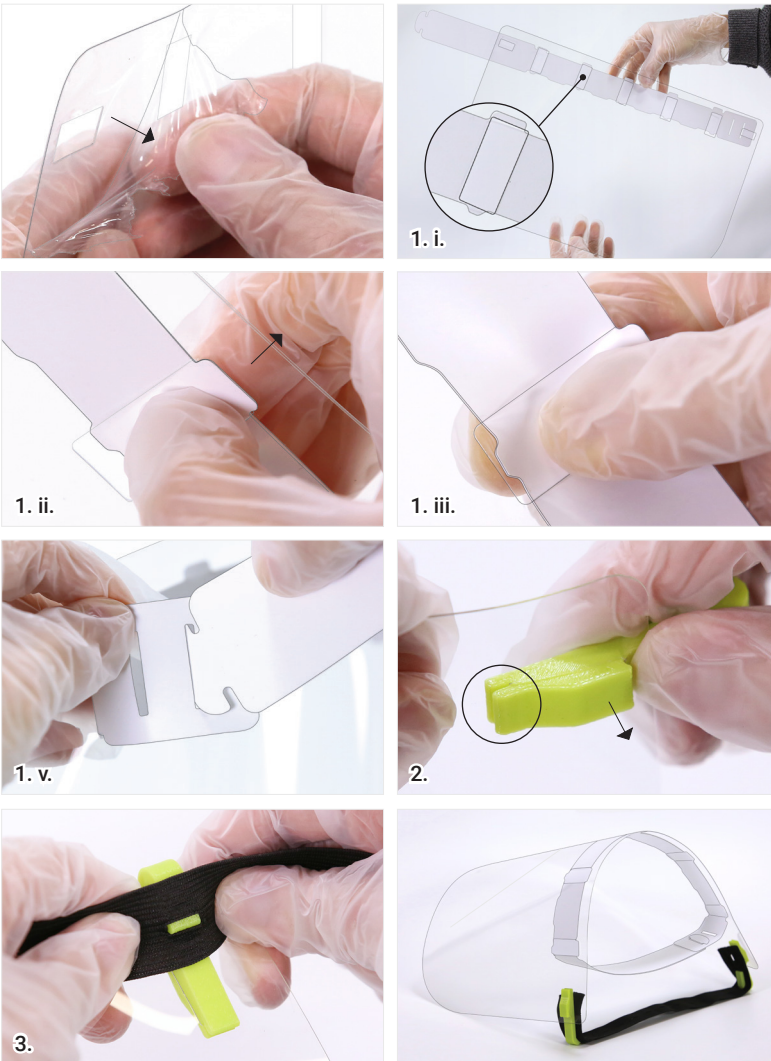
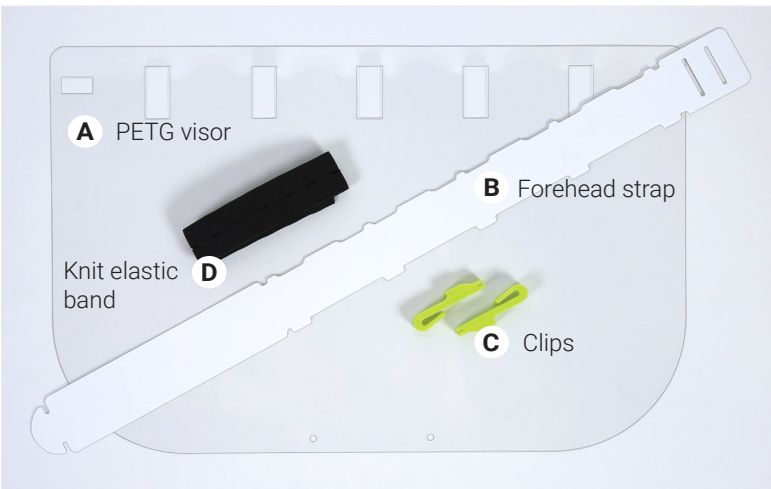
Tinkerine Face Shield Lite: Manual

Tinkerine Face Shield Lite is a **Class I medical device** used to protect the facial area from splashes, sprays, or spatters of bodily fluids. The visor and forehead strap can be sanitized and reused for continued protection.

⚠ BEFORE ASSEMBLY

Make sure the protective films on both sides of the PETG visor are peeled off.

Step 1. Attaching the forehead strap to the visor

- i. Hold the forehead strap underneath the visor in the orientation shown in the picture. Align the strap's five rectangular tabs to the five cutouts on the visor.
- ii. Starting from the centre of the five cutouts, slide the rectangular tab into the cutout.
- iii. Place your thumb over the cutout and your index finger under the trapezoid tab. In a pinching motion, bend the strap until the trapezoid tab snaps into place.
- iv. Repeat for the other four cutouts.
- v. Connect the two ends of the strap to form a loop. Make sure the ends are on the inside of the loop. Hook through the outer slot for a looser fit and the inner slot for a tighter fit.

Step 2. Attaching the clips

- i. From the inside of the visor, slide a clip into the visor's top right corner hole until the tip of the clip grabs onto the visor.
- ii. Pop the clip's centre tab into the visor.
- iii. Repeat the above steps for the left corner hole.

Step 3. Attaching the knit elastic band

- i. Attach the knit elastic band onto the hooks on both sides. It's easier to hold the band perpendicular to the clip as shown in the picture.
- ii. Adjust the band for a snug but comfortable fit.

Reusing the face shield

The visor and the strap can be safely sanitized with isopropyl alcohol (70%) or hydrogen peroxide (28%) before and after every use. Replace the other parts regularly at your own discretion. Replace them immediately if damaged or upon direct contact with anyone with flu-like symptoms. For detailed instructions, please visit tinkerine.com/shield-cleaning-instructions.

WARNING: Tinkerine Face Shield Lite is not rated for impact resistance and fire protection. It helps protect against certain fluid spatters. Misuse may result in sickness or death.

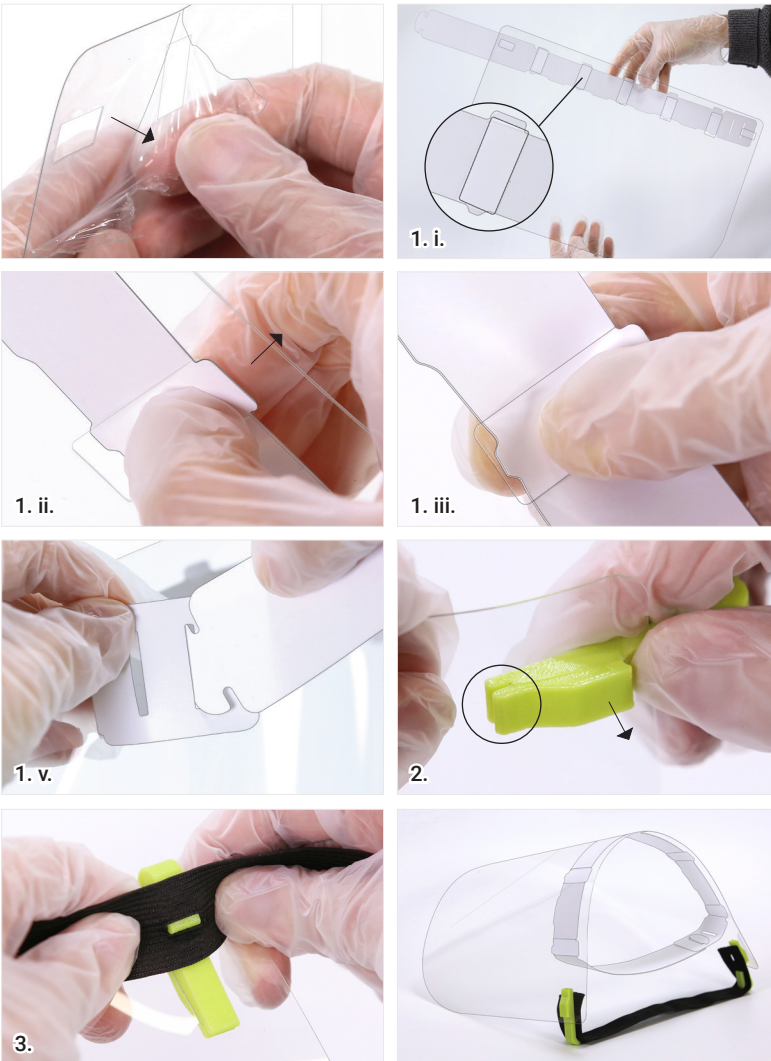
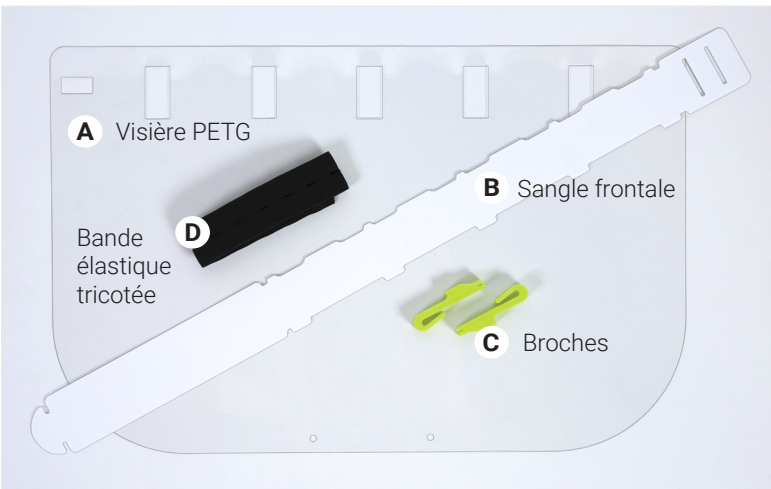
Tinkerine Face Shield Lite is not sterile. Since the conditions and methods of assembly and use of Tinkerine Face Shield Lite are beyond the manufacturer's control, the manufacturer expressly disclaim any and all liability arising from any assembly and use of Tinkerine Face Shield Lite. NO WARRANTY OF FITNESS FOR ANY PARTICULAR PURPOSE, WARRANTY OF MERCHANTABILITY OR ANY OTHER WARRANTY, EXPRESSED OR IMPLIED, IS MADE CONCERNING TINKERINE FACE SHIELD LITE. The user should thoroughly test any application before use. By assembling and using Tinkerine Face Shield Lite, the user does so at their own risk and waives all claims against the manufacturer. The user releases the manufacturer from all liability, whether due to foreseeable or unforeseeable risks, relating to Tinkerine Face Shield Lite. The user is solely responsible for their assembly and use of the Tinkerine Face Shield Lite. The manufacturer is not responsible for errors or omissions relating to Tinkerine Face Shield Lite. The manufacturer does not provide any medical advice.

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Veillez visiter tinkerine.com/shield-cleaning-instructions.

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