Cleaning Tinkerine Face Shield (Pro)

Step 1. Precautions for cleaning

- Before you begin, protect yourself by wearing gloves, a face mask, and another face shield if possible.
- Clean and disinfect the working surface thoroughly.
- Only clean one face shield at a time to avoid cross-contamination.

Step 2. Disassembling the face shield

- Disassemble the face shield in the reverse order of the assembly:
 - i. Remove the lower bracket
 - ii. Remove the forehead cushion
 - iii. Remove the knit elastic band
 - iv. Remove the clips
- The foam cushion, knit elastic band, lower bracket, and clips are reusable up to 10 times for non-medical use.

Step 3. Cleaning the PETG visor


- It is recommended to fully submerge the PETG visor in a disinfectant solution to ensure complete sanitization before and after every use.
- After removing the visor from the solution, allow a few minutes for the visor to dry. Wipe down any remaining disinfectant droplets with a clean paper towel.
- If submerging is impossible, thoroughly spraying the visor with a disinfectant can also be effective. Allow a few minutes to dry. Wipe down any remaining droplets.
- Our recommended disinfectants are:
 - Isopropyl alcohol (70%)
 - Hydrogen peroxide (0.5% - 25%)
- To find out more about specific cleaning products, please refer to the [list of disinfectants approved by Health Canada](#).

Step 4. Reapply anti-fog coating

Reapply TruSight Anti-Fog Solution to the inside of the visor.

Step 5. Reassembling the face shield

To reassemble the face shield, refer to the assembly manual.

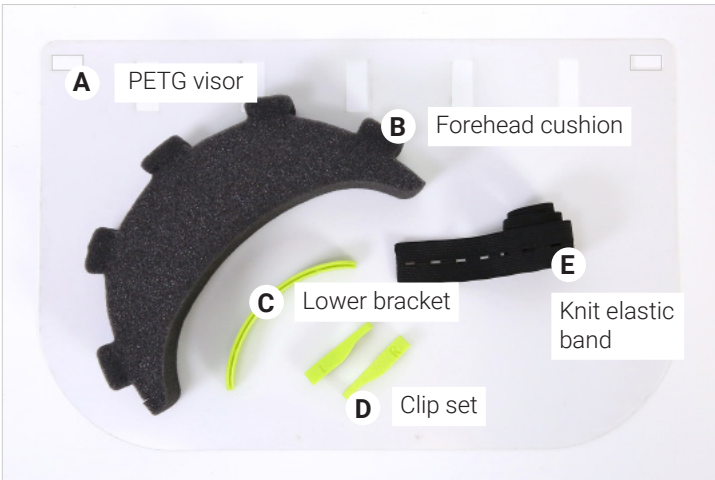Tinkerine Face Shield is not sterile. Since the conditions and methods of assembly and use of Tinkerine Face Shield are beyond the manufacturer's control, the manufacturer expressly disclaim any and all liability arising from any assembly and use of Tinkerine Face Shield. NO WARRANTY OF FITNESS FOR ANY PARTICULAR PURPOSE, WARRANTY OF MERCHANTABILITY OR ANY OTHER WARRANTY, EXPRESSED OR IMPLIED, IS MADE CONCERNING TINKERINE FACE SHIELD. The user should thoroughly test any application before use. By assembling and using Tinkerine Face Shield, the user does so at their own risk and waives all claims against the manufacturer. The user releases the manufacturer from all liability, whether due to foreseeable or unforeseeable risks, relating to Tinkerine Face Shield. The user is solely responsible for their assembly and use of the Tinkerine Face Shield. The manufacturer is not responsible for errors or omissions relating to Tinkerine Face Shield. The manufacturer does not provide any medical advice.

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